TWIN

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INDICATIONS FOR USE

Pyrogen-free, sterile, blue, temporary tissue dye, intended for staining and visualization of the retinal membranes and the inner limiting membrane. **TWIN** is intended for use by centres performing surgeries. It should be used by qualified medical personnel only. Any other use must be considered improper.

INDICATIONS FOR USE
Pyrogen-free, sterile, blue, temporary tissue dye, intended for staining and visualization of the retinal membranes and the inner limiting membrane. **TWIN** is intended for use by centres performing surgeries. It should be used by qualified medical personnel only. Any other use must be considered improper.

COMPOSITION AND PACKAGING

TWIN, 0.18% tripan blue and 0.03% blue dye with formula C48H50N3NaO7S + 4% Ficol® 400 in physiological solution. Each box contains one instruction sheet and labels for product traceability.

DIRECTIONS FOR USE

- Make sure that the packaging is intact.
- Remove the plunger, syringe, and backstop from the pouch.
- Insert the plunger in the rubber stopper of the syringe and screw it clockwise.
- Insert the backstop into the syringe wings.
- Remove the syringe cap and attach a disposable cannula to the syringe.
- Before injection, purge the syringe, verifying the correct sliding of the stopper into the syringe.
- Apply a few drops of **TWIN** on the tissue to be visualized. Do not mix with any other substances.
- Remove any excess **TWIN** with saline solution.
- Properly discard any remaining **TWIN**.

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CONTRA-INDICATIONS

- Product is contra-indicated in patients with known hypersensitivity to the declared ingredients.
- Do not use in pregnant women, nursing women, women of childbearing age who could become pregnant, and children.
- Product is contra-indicated in patients with hydrophilic acrylic intraocular lenses.

WARNINGS AND PRECAUTIONS

- Before use, store as indicated on label.
- Single Use Only – Do not resterilize. Use or sterilization of single use products that have been already used may cause cross-contamination.
- The integrity of the product is warranted only if the package is intact – Do not use if the package is open or damaged.
- The expiry date is shown on the product label – Do not use after the expiry date.
- Do not use in the presence of sediment.
- Do not use cold solutions during surgery.
- Do not use together with other substances.
- Soon after application to the tissue, it is advisable to remove any excess **TWIN** by thorough rinse with saline solution.
- Avoid allowing **TWIN** to get under the retina.
- Do not apply **TWIN** to a macular hole if the inner limiting membrane has already been removed.
- TWIN** should only be used by qualified medical surgeons, in an operating theatre, surgery, or other suitable hospital environments. Any residues must be disposed of as hospital waste material. In case of accidental breakage of syringe before or during use, confine the spill and absorb it in inert material and dispose of in compliance with procedures in force.
- Dispose of empty packages in compliance with regulations in force.

SIDE EFFECTS

Following surgeries with the use of dyes, inflammatory reactions, bleeding, transient increase in intraocular pressure and progression of cataract may occur.
Use of tripan blue has been associated with an increased rate of cystoid macular oedema.
Short-term in vitro studies and experimental studies have shown no toxicity of the dye. Retinal and sub-retinal tissue changes over time cannot be ruled out following the use of **TWIN**.
Although no allergic reactions to the dye are known, **TWIN** may trigger allergic reactions in sensitive individuals.

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STORAGE

TWIN should be stored at room temperature.

For any report on this product, quote the lot number printed on package. For any comments and suggestions, go to www.alchimiasrl.com

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AL.CHI.M.I.A. S.r.l.