|                    |           |
|--------------------|-----------|
| <b>TRYPAN BLUE</b> | <b>TR</b> |
|--------------------|-----------|

#### Açıklama:

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#### Description:

Ocublu Try Trypan Blue / Nanoblu Trypan Blue is a sterile tissue dye for anterior segment surgery. Trypan Blue has been developed as an auxiliary device for performing capsulorhexis during cataract surgery. Trypan Blue is used to selectively stain the anterior lens capsule, which helps with facilitating capsulorhexis and minimizing the risk of tearing during cataract surgery.

#### Composition

1 mL Trypan Blue Contains:

|                            |                            |                            |
|----------------------------|----------------------------|----------------------------|
| 0,2 mg of Trypan Blue      | 0,4 mg of Trypan Blue      | 0,6 mg of Trypan Blue      |
| Sodium chloride            | Sodium chloride            | Sodium chloride            |
| Disodium hydrogenphosphate | Disodium hydrogenphosphate | Disodium hydrogenphosphate |
| Sodium dihydrogenphosphate | Sodium dihydrogenphosphate | Sodium dihydrogenphosphate |
| Water for injection        | Water for injection        | Water for injection        |

#### Instructions For Use :

This product is designed for use by ophthalmologists only. Trypan Blue is injected into the anterior chamber using a tuberculin syringe between the air bubble and the lens capsule. Trypan Blue is achieved as soon as the dye comes into contact with the anterior lens capsule. The anterior chamber is then flushed to remove any residual dye. A viscoelastic solution is then injected. The cataract operation is then performed in the usual manner.

#### Indications

Trypan Blue is indicated for intraocular use for staining the capsular tissues during ophthalmic surgery, especially involving cataract extraction using phacoemulsifications. Trypan Blue permits clear visualization of the anterior capsule in contrast to non-stained lens cortex. It is intended for intraocular use only.

#### Contraindications

No contraindications are known when used in accordance with recommendations.

#### Side Effects

Side effects are not known.

#### Presentation and Sterilisation

Trypan Blue is available in vial or syringe containing 1.0 ml of sterile solution. Each box contains 10 units. It is sterilized using steam.

#### Storage Conditions

Trypan Blue must be stored at 10 - 30°C, protected from light.

Shelf life : 3 years

#### Warnings and Precautions for use

Product is only intended for single intraocular application only, it should not be administered by intravenous and intramuscular routes. Do not resterilize. Do not use this product after the expiry date. Any sterile pack that has been opened and/or damaged must not be used. The medical device must be removed from its primary package aseptically. After use, dispose of the product with medical waste in accordance with local waste management regulations.

#### Adverse Reactions

Unexpected side effects and complications related to product should be reported to the competent authority of the member state and to ATAKAN DEDE-MİRAY MEDİKAL.

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