
EN	ENGLISH
----	---------

OPHTHALMIC CANNULAS

Instructions for Use

DESCRIPTION

Ophthalmic cannulas consist of micro-tubes based on a hub with Luer Lock thread that can be attached on a handpiece or syringe. The tubes end in tips which have a plethora of shapes and sizes according to their intended use.

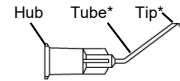(* Shapes vary per product description)

Materials

Hub: Polypropylene (colour-coded per Gauge according to ISO 6009)

Tube: Stainless Steel

Packaging

Each ophthalmic cannula is sterilized by ETO and individually packed in a pouch to ensure sterility; marketed in boxes of 20 pcs.

INTENDED USE

The cannula tip is inserted through micro-incision into eye cavities or the surrounding area, assisting surgeons to perform certain tasks (e.g. the delivery or removal of fluids).

CLINICAL BENEFIT

Being used in a wide array of ophthalmic procedures, they provide tailored solutions for different surgical needs.

Also, the use of an ophthalmic cannula provides surgeons with precision, control, and safety during various ophthalmic surgeries by facilitating controlled delivery, removal, and irrigation of fluids and viscoelastic devices within the delicate eye structures. This leads to better surgical outcomes by enabling precise maintenance of intraocular pressure (IOP), preventing corneal trauma and reducing fluid turbulence.

INTENDED PATIENT POPULATION

Patients of all ages.

INTENDED USERS

Trained and qualified ophthalmic surgeons only.

INDICATIONS

A variety of ophthalmic surgical procedures, such as cataract, refractive, vitreoretinal and glaucoma operations.

CONTRAINDICATIONS

Not to be used for a purpose different than the intended one. Not be used with medication that reacts with plastic.

PRECAUTIONS

- Read the instructions before use.
- Care should be taken to avoid damage of the needle tip.
- Some needle tips are very sharp. Handle with safety till their disposal.

⚠ WARNINGS

- Single use only. Do not re-use.
- Do not re-sterilize
- Do not use it after expiry date.

- Do not use it if product's aseptic condition is considered spoiled by broken package.

REPORTING

Adverse events and/or potentially sight-threatening complications that may reasonably be regarded as product related should be reported to the manufacturer.

STORAGE

Store away from direct sunlight, in a clean and dry place, at temperatures between 0 and 45°C.

FR	F R A N Ç A I S
----	-----------------

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