MELAfol®**MELAfol®****MELAfol®****MELAfol®****MELAfol®****MELAfol®****MELAfol®****Gebrauchsanweisung**

PRODUKTNAMEN:

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INSTRUCTIONS FOR USE:

1. MELAfol® pouches ready for use. MELAfol® pouches need to be cut to the required length and sealed at one end to create a pouch. Do not use MELAfol® reels and pouches for reuse. The MELAfol® reels and pouches are not included for reuse.
2. Cover sharp ends of instruments with a sterile protective, to prevent puncturing or cutting.
3. Place the items that is to be sterilized into the pouch. Do not exceed more than 75 % of their capacity.
4. Always leave a minimum distance of 1.5 cm between the material and the individual ends to avoid an even air flow.
5. Manually remove as much air as possible from the pouches before sealing. Seal the pouches calibrated and validated sealing device according to the manufacturer's recommendations. Using the recommended temperature of the device must be set to 180 °C. After sealing the pouch should fit itself during peeling from the foil.
6. Avoid stressing the sealing seams, e.g. by filling the pouches horizontally or at flat surface.
7. When sealing the pouches, make sure that the basic way is that the film follows to film and paper faces to paper.
8. Place sharp edges of instruments upright in the pouch to prevent them from being cut.
9. Place sharp edges of instruments so that there is enough space between the sterilization pouches so that a hand can slide between them.
10. Process the load according to the sterilizer.
11. Hand pouches with care when removing from the sterilizer to prevent them from falling in an area protected from sunlight, water and other liquids.
12. Do not open a wet pouch. Repeat the sterilization process again after changing the pouch.
13. To open a pouch, hold the plastic side with the thumb and index finger of one hand. Pull off the paper side with the other hand. Always open the pouch from the bottom of the arrow. For a convenient convenient open the pouch slowly.

STORAGE INSTRUCTIONS:

Store MELAfol® at max. +35 °C and max. 70 % humidity. Sterilization reels and pouches have a shelf life of 5 years when stored at recommended conditions.

Store MELAfol® reels and pouches away from moisture, excessive air flow and direct sunlight. Do not use MELAfol® reels and pouches once they have become damp.

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