


Orizon

Preloaded

Hydrophobic Acrylic
Foldable Intraocular Lens


Instructions for use

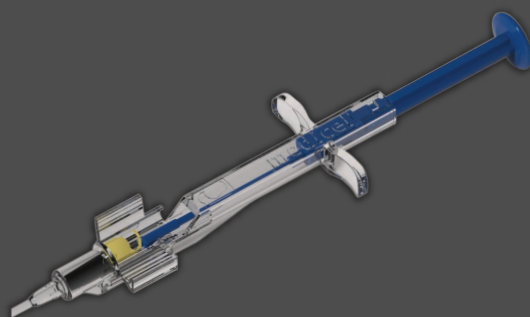


Orizon

Preloaded Hydrophobic Acrylic
Foldable Intraocular Lens

Instructions for use

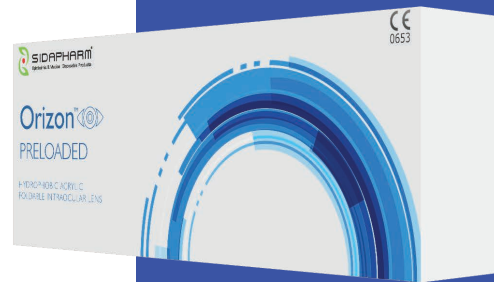


Carefully remove the system from the packaging and hold it horizontally. ORIZON is already preloaded with Sidapharm's premium intraocular lenses

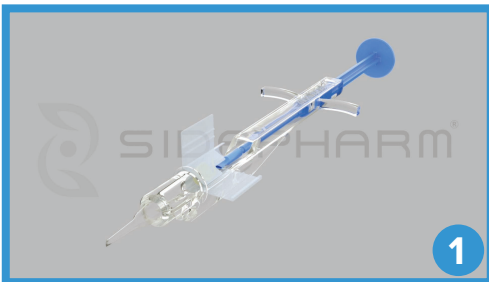


Add a small quantity of viscoelastic into the nozzle at the front to ensure smooth delivery throughout the process

Carefully add sufficient quantity of viscoelastic below the lens

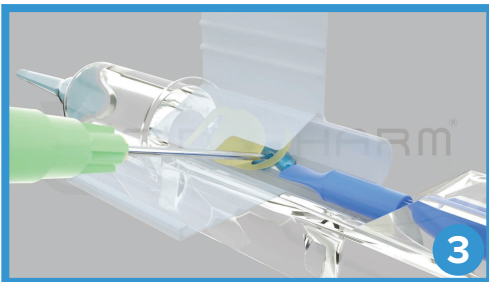



In addition place viscoelastic over the lens and below the flap as shown in the picture

Slowly press the top of the cartridge lid until it closes entirely, ensuring that the lens stays in place.

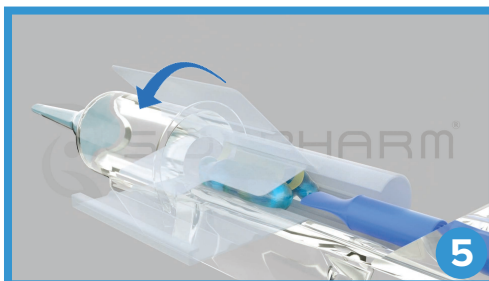


Press the cartridge lid firmly until you hear a "click". The lens is secured inside and ready for implantation

Make sure that the blue plunger is straight and located inside the canal and gently push the lens to ensure it moves unhindered within the rail

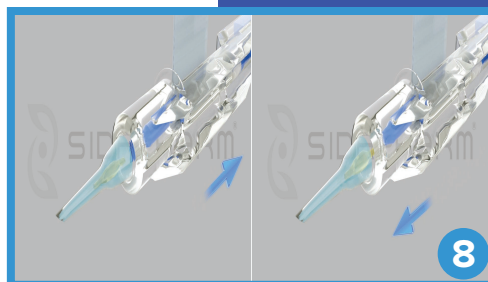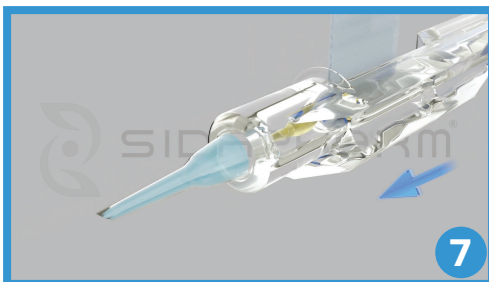

Proceed by slightly pushing and retracting the plunger (about 1-2 mm each time), to allow the haptics to position correctly inside the rail

As the optic emerges from the nozzle allow a few seconds for the lens to unfold and give one final push to implant it inside the eye

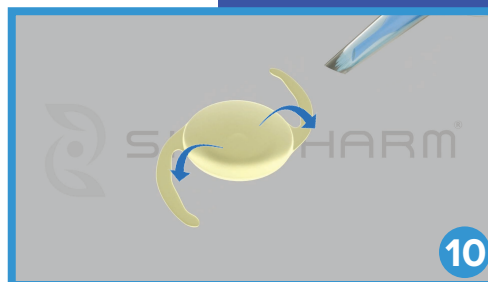

The lens unfolds controllably and smoothly as it is implanted