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(EN) Instructions for use - Bowie-Dick disposable test pack

The stericlin® Bowie-Dick disposable test pack is used to carry out functional tests on steam sterilisers. It is based on an indicator colour system that contains no metals in any form. The indicator system is odourless and does not stain. To ensure that the sensitive indicator functions properly, the instructions for use must be followed carefully. The indicator system complies with ISO 11140-4, the indicator sheet is a type 2 indicator in accordance with ISO 11140-1.

1. Preheat the steriliser using an empty batch before carrying out the test.
2. Place the stericlin® Bowie-Dick disposable test pack horizontally at the bottom of the steriliser, e.g. on the lowest level of the loading trolley or above the drain of the preheated, empty steriliser. The pack must be inserted so that the printed label is on the underside.
3. Start the Bowie-Dick programme (134°C. for 3.5 minutes *). After completion, the test pack must be removed from the steriliser immediately, opened, the test sheet checked, analysed and the result documented.
4. To check the result, place the sterilised sheet on a light-coloured surface. Please note: Looking at the sheet against a light source is not suitable for evaluating the result. Compare the colour of the indicator at the edge of the sheet with the colour of the indicator in the middle of the sheet.

If the steriliser has achieved proper steam penetration and the steam quality was sufficient, the indicator will be transformed completely, i.e. uniformly, from blue to pink.

If the steriliser has not achieved proper steam penetration or the steam quality was insufficient, the indicator shows an area in the centre that is not or not completely transformed. In this case, check your steriliser.

5. The documentation of results with signature and countersignature (supervisor) is legally correct and sufficient.

*) Temperature and time are understood to be within the usual process tolerances, see e.g. EN 285 and DIN EN 13060.

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