THIS ARTWORK IS PROPERTY OF:

SIFI

PRODUCT NAME:

Instruction For Use Easyring

PRODUCT CODE:

F41029V2018-03

COUNTRY:

ITA - UK - FRA

1° Draft: 12/03/2018

SEPARATIONS: **(Black - Fustella)**

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BARCODE TYPES:

MEASURES (mm): **420x297**

MATERIAL: g/sq.mt:

Template:

FRONTE

ISSUE DATE

14-03-18

OPERATOR

G. Longo

DRAFT

04

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Fabbricante:

MorcherGmbH

Kapuzinerweg 12

D-70374 Stuttgart (Germania)

Distributore:

SIFI S.p.A.

Via Ercole Patti, 36

95025 Laviniaio – Aci S'Antonio (CT) – Italia

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95025 Laviniaio – Aci S'Antonio (CT) – Italy

Product description

The EASYRING™ consists of a Capsular Tension Ring and an Injector for Single Use.

Capsular Tension Rings, hereafter referred to only as implants, are sterile, non-optical implants for expansion of the capsular bag after cataract extraction.

It's made from one piece of PMMA (polymethyl methacrylate).

The Injector for Single Use, hereafter referred to only as injector, is an auxiliary device, which simplifies the implantation of the Capsular Tension Ring.

The eyelet of the implant is already hooked in the injector and can be loaded in the injector immediately after removal from its package, it is then ready for implanting.

Warnings

- Do not use the EASYRING™ if the sterilized package is open or damaged
- Do not re-sterilize the implant or the injector by any method
- Do not reuse the implant or the injector
- Rinse the implant only with sterile intraocular rinsing solutions such as sterile Ringer's solution or sterile BSS® solution
- Store only at room temperature
- The injector must only be used with the implant provided with it

Cautionary measures

A high level of surgical skill is required for proper implantation. The surgeon should have observed and/or assisted in numerous implantations and successfully completed one or more appropriate courses before attempting to perform implantation.

The surgeon performing the implantation must read all the material provided by SIFI S.p.A. for the correct handling and insertion of this implant, before he performs the implantation.

Indications

The Capsular Tension Ring stabilizes the capsular bag in cases of:

- In cases of pseudoexfoliation
- In cases of defective or missing zonules
- In cases of severe myopia
- In cases of zonulolysis
- In cases of Marchesani's Syndrome
- Primary Zonular weakness
- Secondary Zonular weakness (e.g. Marfan-Syndrome)

Advantages of the implant

- When there is possible IOL luxation
- When complicated surgical conditions are anticipated
- To prevent one-sided capsular bag shrinkage
- When IOL implantation may be difficult
- For circular expansion of the capsular bag

Contraindications

These include the usual contraindications for eye surgery, as well as the following:

- Microphthalmia
- Macrophthalmus
- Damaged capsular bag
- ICCE- technique
- Infants up to 12 months old
- Chronic uveitis
- Shallow anterior chamber
- Progressive eye disorders (proliferative diabetic retinopathy, uncontrolled glaucoma)
- Pre-operatively detected complications (such as vitreous prolapse, vitreous hemorrhage)
- Patients with a shallow anterior segment, for example, with microphthalmia or certain forms of chronic angle closure glaucoma
- Plate haptic IOL, since the capsular adhesion required for the fixation of the IOL is prevented by the implant (danger of rotation and tilting out of position, as well as luxation after possible YAG laser capsulotomy)
- Rupture of the lens capsule, with or without vitreous prolapse
- Persistent bleeding or other factors obstructing visibility

Side-effects

Side-effects due to the application of EASYRING™ are not known at this time

Packaging and expiration date

- EASYRING™ is sterilized by gamma radiation
- EASYRING™ is supplied sterile and dry in a germproof, sealed primary container (blister packaging) that is, in turn, packed in a sterile blister package (sterile double packaging)
- Sterility is guaranteed only when the blister packages are neither opened nor damaged
- Expiration date is indicated on the folding box and the primary container
- The EASYRING™ should not be used after the expiration date indicated!

Storage conditions / Transport conditions

- Store flat at room temperature
- Do not expose either to direct sunlight or to extreme temperatures
- Protect from moisture / water and keep dry

Patient information

- Each patient should be well-informed about the implant before implantation.

THIS ARTWORK IS PROPERTY OF:

SIFI

PRODUCT NAME:

Instruction For Use Easyring

PRODUCT CODE:

F41029V2018-03

COUNTRY:

ITA - UK - FRA1st Draft: 12/03/2018SEPARATIONS: **(Black - Fustella)**

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Conditions de conservation/conditions de transport

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- The surgeon performing the implantation must inform the patient of all known side-effects and risks.
- The patient must be instructed to inform the doctor in charge completely of all side-effects after implantation.

Patient card

The relevant details must be entered on the front side of the enclosed patient card, and one of the enclosed labels, containing details of the implant, should be affixed to the back of the card. The implantation date should be written on this label, and the operated eye should be marked. This card is given to the patient, who should look after it carefully and present it to any eye specialist that he/she may see in the future.

Notes on handling

- Before opening the box, check the details on all labels, concerning the type, specific data and expiration date.
- Under aseptic conditions, remove the implant as follows:
 - open the outer blister packaging at the positions marked and remove the primary container
 - carefully pull off the sealing of the second blister packaging (primary container)
 - carefully remove EASYRING™ together with the hooked ring.
- Inspect the implant before using, only use an undamaged implant.
- Please, remove the clip at the plunger of the injector only after the implant has been withdrawn completely into the cannula of the injector.

Notes on the implantation

Caution: If not handled correctly, perforation of the capsular bag or damage to zonular fibers is possible! Eyes with pseudoexfoliation syndrome and decreased anterior chamber depth show a greater risk of zonular defects or other intraoperative complications.

Sufficient high-viscosity viscoelastic material must be used during implantation.

To make the implantation easier, dimension the capsulorrhexis as large as possible. This will also help to minimize any risk of complications with the implant in the case of pseudoexfoliation-syndrome and zonular weakness.

The implantation should be performed as much as possible in a tangential direction, to avoid potential snagging of the leading loop. If necessary, the implant may need to be pulled back slightly and the capsular bag fornix expanded by the subsequent injection of high-viscosity viscoelastic.

Please note: the inscription on the injector should be legible from above during implantation. This way the injector is in the right position and the implant can be released from the injector.

Introduce the tip of the injector in the capsular bag. Carefully depress the plunger to implant the Capsular Tension Ring into the capsular bag.

Make sure, it is positioned correctly and that the plunger is in its fully extended position. Then release the Capsular Tension Ring from the injector by removing the hook of the injector carefully from the eyelet of the Capsular Tension Ring. Then withdraw the hook back into the injector. This way you will not damage any tissue when removing the injector.

Caution: An implant which was damaged during the implantation must not remain in the eye!

Important notes

Notes for laser treatment:

Focus the laser beam precisely on the action site behind the implant. A laser beam focused on the implant itself will lead to a damage of the implant.

Exclusion of liability

SIFI S.p.A. and MORCHER® GmbH cannot be held liable or responsible for either the insertion procedure or the surgical technique employed by the surgeon. Selection of an implant appropriate for a particular patient and that patient's condition is solely the responsibility of the surgeon in charge.

Return of a damaged / unsterile implant

When an implant is returned, the serial number, batch designation and the reason for its return must be specified. The packaging for the returned item should be chosen sensibly, in accordance with the condition of the implant, and if necessary, any details of the state of contamination should be included.

Adverse Reaction Reporting

Adverse reactions and/or potential sight threatening complications that may reasonably be regarded as implant related and that were not previously anticipated in nature, severity or degree of the incidence, must be reported to SIFI S.p.A..

Such information is requested from all surgeons, in order to document potential long-term effects of the use of the implant. To report such adverse reactions/complications, please contact:

SIFI S.p.A.

www.sifigroup.com - customer.surgical@sifigroup.com

Freephone: +39 800 348 122

Abbreviations and packaging symbols

ABBREVIATIONS / SYMBOL	MEANING
PC	Posterior chamber
D	Diopters in aqueous humor
Ø _b	Body diameter
Ø _t	Total diameter
SN	Serial number
	Batch code
	Use by: Year - Month
	Sterilized using irradiation
	Do not re-use
	Caution, consult accompanying documents
	Keep dry
	Keep away from sunlight
	Do Not Resterilize
	Recyclable Packaging
	Do not use if package is damaged
	Consult instructions for use

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