IT **CE** **0477**

HPF
Perfluorocarburi "High purity"

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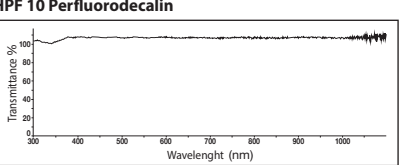
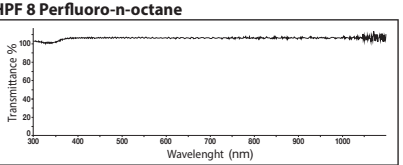
Transmittance (300-1100 nm) vs air

INDICATIONS FOR USE
Intraoperative intraocular tamponade during vitreoretinal surgery. Any other use must be considered improper. The safety and effectiveness of the product in patients under 15 months of age has not been established.

DIRECTIONS FOR USE
Make sure that the package is intact. For each variant of the product, follow the directions for use below.

Vials

- Remove the vial from the pouch in the sterile field.
- Under aseptic conditions, transfer the content of the vial to a sterile device suitable for injection, taking care not to introduce air bubbles.
- Do not mix with other substances.
- Inject the product slowly according to the accepted medical practice.

Syringes

- Remove the syringe from the pouch in the sterile field.
- Screw the plunger rod into the rubber stopper of the syringe.
- Attach a cannula of appropriate size to the syringe.
- Make sure that the luer lock adapter is firmly attached to the syringe.
- PURGE, KEEPING THE SYRINGE IN A VERTICAL POSITION WITH THE CANNULA FACING UP AND PUSHING OUT APPROXIMATELY 0.5 ML OF FLUID, VERIFYING THE CORRECT SLIDING OF THE RUBBER STOPPER INTO THE SYRINGE.
- Inject the product slowly according to the accepted medical practice.

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IF SILICONE OIL IS USED DURING THE SAME SURGERY, PERFORM FLUID-AIR EXCHANGES TO ENSURE COMPLETE REMOVAL OF PERFLUOROCARBONS, BEFORE INJECTING SILICONE OIL.

CONTRAINDICATIONS

- Perfluorocarbons are relatively inert substances; nevertheless it is recommended that they are not used in patients with known hypersensitivity to the declared ingredients.
- Perfluorocarbons are contraindicated for long-term intraocular use or as permanent vitreous substitutes.

WARNINGS AND PRECAUTIONS

- Before use, store as indicated on label.
- Integrity of products is warranted only if package is intact. Do not use if package is open or damaged or any component is received in a defective condition.
- The expiry date is printed on product label. Do not use after the expiry date.
- Do not shake and do not mix with any other substances. The concomitant use of perfluorocarbons and corticosteroids may cause the appearance of a precipitate into the eye.
- Before injecting the product, make sure that the luer lock adapter is firmly attached to the syringe.
- BEFORE INJECTING THE PRODUCT, PURGE THE SYRINGE KEEPING IT IN A VERTICAL POSITION WITH THE CANNULA FACING UP AND PUSHING OUT APPROXIMATELY 0.5 ML OF FLUID, VERIFYING THE CORRECT SLIDING OF THE RUBBER STOPPER INTO THE SYRINGE.
- Products shall be used by medical personnel only in surgery room. Any residues must be disposed of as sanitary waste. In case of accidental leakage or spillage before or during use, confine the spill and absorb in inert material (e.g. sand, absorbent granules, etc.), place this material in a plastic container and dispose of in compliance with procedures for waste.
- Do not inject directly into the vitreous, or concomitantly with vitreous aspiration.
- Subretinal migration may occur during product injection. Special care should be taken to examine for subretinal migration of the product.
- PERFLUOROCARBONS SHOULD NOT BE INJECTED DIRECTLY INTO THE VORTEX VEIN. IN ADDITION,

ADVERTENT INGRESS OF INTRAOCULAR PERFLUOROCARBONS INTO THE VORTEX VEIN SHOULD BE AVOIDED.

- SPECIAL PRECAUTIONS ARE INDICATED WHEN PERFLUOROCARBONS MAY ENTER THE VASCULAR CIRCULATION (E.G., DURING ENDORESECTION OF INTRAOCULAR TUMORS AND OCULAR TRAUMA). RARE CASES OF AIR EMBOLISMS POTENTIALLY RELATED TO THE PFCLs INGRESS HAVE BEEN REPORTED.**

- IN THE LITERATURE, THE PERMANENCE OF PERFLUOROCARBON RESIDUES HAS BEEN ASSOCIATED WITH SILICONE OIL STICKINESS AND FLOATING. PERFORM FLUID-AIR EXCHANGES TO ENSURE COMPLETE REMOVAL OF PERFLUOROCARBONS, BEFORE INJECTING SILICONE OIL.**

- Single use only – Do not resterilize. Use or sterilization of single use products that have been already used may cause cross-contamination.
- Fluids extracted from the eye are potentially bio-hazardous and constitute a necessary disposal and dispose of in compliance with national regulations on workplace safety and quality assurance principles.
- Dispose of empty packages in compliance with regulations in force.

SIDE EFFECTS

- Temporary elevated intraocular pressure, hypotony, glaucoma, cataract and corneal, anterior chamber, iris anomaly or fatal air embolism after uneventful choroidal endoresection without air infusion can occur after surgery.
- Long-term intraocular perfluorocarbons (> 2 weeks) have been associated with mechanical damage to the retina and toxicity; remove completely product from the eye at the end of surgery.

STERILIZATION

Vials and syringes are sterilized by moist heat. Single use only – Do not resterilize.

STORAGE

Store at room temperature. Keep away from heat and direct light.

For any report on this product, quote the lot number printed on the label. For any comments and suggestions, go to www.alchimiasrl.com

MANUFACTURER
ALCHIMIA S.r.l.
Viale Austria 14
35020 Ponte S. Nicolò (PD) - Italy
e-mail: info@alchimiasrl.com

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